everon

Removing a Pendant or Sensor





Place the item to be removed near its home base station (removing requires both the item to be removed and the hub to be close to each other. If they are not close to each other, this process will be unsuccessful).

Log in to the portal (see User guide – "Portal access")

Click on the **Device Management** section to the left in the main menu.

9	everon"	Dashboard
♠	Dashboard	Customer info
Ð	Device management	veron UK demo (C0250A01) Turkey Mill Jaidstone
2	Staff members	nN23 1HU Maidstone Kent
	Receiver groups	Enabled features

You will then see a list of all your locations/sites/schemes.

Scan down until you see the location/site/scheme where you wish to remove the item from.

Click on the location/site/scheme and this expands it so you can see all dwellings/flats there.

Everon UK Number of bases: 4 / Number of devices: 8						
	Serial number	Device name	Device type			
0	3130357B	Flat 1	Base station (TP-202)			
0	313059F5	Flat 2	Base station (TP-202G)			
∨ ⊚	31305519 052CC34E	Flat 3 Pendant test 1	Base station (TP-202G) Monitoring button (PL-100G2)			
0	132E2D35	Pull test 1	Controlswitch (PL-105F)			



Click on the item you wish to remove, and then to the right side, this screen opens: -

Ľ	Pendant test 1 (052CC34E)
	Information Settings Device management
Device type	Replace device Please enter the serial number of the replacement device
Base station (TP-202)	ID ₹ Replace
Base station (TP-202G)	Unpair device
Base station (TP-202G)	Unpair this device from the system by clicking the button. The device will be unpaired after the confirmation and it cannot transfer a paired to a base.
Monitoring button (PL-100G2)	Delete
Controlswitch (PL-105F)	
Base station (TP-202G) Monitoring button (PL-100G2)	Update device firmware Current firmware: 50
Controlswitch (PL-105F)	Available firmware versions

Click on the Device management tab: -

Click the "Delete" button.

It will ask you if you are sure you want to delete – and click on the "Delete button" again.

You need to now listen to the hub and you will hear it play a single tone to confirm that the item has been removed.

습 Everon UK			Ľ		Pendant test 1 (052CC34E)			
Numbe	er of bases: 4 / Number o	f devices: 8				Information Settings Device management		
	Serial number	Device name	Device type			Serial number: 052CC34E Name: Pendant test 1 Location: Eveno IVK (C22CDE01) Added to system: 28/2/2023 15:37:22	Activation (Alert routing
0	3130357B	Flat 1	Base station (TP-202)				Non-movement alarm	No alert routing
0	313059F5	Flat 2	Base station (TP-202G)		Software version: 30 Last connection registered: 7/6/2023 13:16:03 Battery status:	Alarm button	Range lest	
~	31305519	Flat 3	Base station (TP-202G)			🖉 Modify	Fall alarm	No elect routing
0	052CC34E	Pendant test 1	Monitoring button (PL-100G2)					No distributing
0	132E2D35	Pull test 1	Controlswitch (PL-105F)			Long press	No alert routing	



To check the item has been removed, refresh (press F5) your browser and look again at the devices at the address, you will see the item is no longer there.

Everon UK Number of bases: 4 / Number of devices: 8						
	Serial number	Device name	Device type	Last connection registered		
0	3130357B	Flat 1	Base station (TP-202)	≤ 5/9/2022 15:16:29		
0	313059F5	Flat 2	Base station (TP-202G)	-		
× ٥	31305519 132E2D35	Flat 3 Pull test 1	Base station (TP-202G) Controlswitch (PL-105F)	 ★ 6/4/2023 10:01:39 ★ 6/4/2023 10:12:00 		

You can now log out of the portal.