

Night Owl Alarm Testing



Contents

| | |
|---------------|---|
| Alarm testing | 3 |
| How to test | 3 |



Alarm testing

Before testing alarms, please check the following system settings:

- Run AMS in the room and check that all zones are correct.
- Check that the alarm type you are about to test is enabled in the Night Owl portal and that it is also alarm routed to the correct receiver group in the Everon portal.
- If there is a schedule set for the alarm type you are about to test, make sure that the alarm is active now or discard the schedule and recreate it after your test.
- Please ensure that you are logged in to the correct receiver group in the EMA application, and that the Night Owl is not in a paused state, either through the QR-code, or that presence is marked through the NFC-tag.

How to test

- **Fall alarm**

Stand up straight and lie down on the floor. After about 20-60 seconds, an alarm will be sent from the Night Owl.

- **Upright in bed and Out of bed**

Lie down in bed and stay for at least 90 seconds. Get up and leave the bed. Alarm is sent from the Night Owl.

- **Leaving chair**

Sit in the chair and wait at least 90 seconds. Get up and leave the chair. Alarm is sent from the Night Owl.

